

VPAP ADAPT SV
menu

SETTINGS
enter ↓ exit

ASV MODE

CPAP MODE

MASK: FULL FACE
change ↓ exit

LEARN CIRCUIT?
yes ↑ exit

SMARTSTOP: OFF
change ↑ exit

SOUND LEVEL: LOW
change ↑ exit

RAMP: 20min
change ↓ exit

MASK: FULL FACE
change ↑ exit

LEARN CIRCUIT?
yes ↑ exit

SMARTSTOP: OFF
change ↑ exit

SOUND LEVEL: LOW
change ↑ exit

RESULTS
enter ↑ exit

MASK FIT: ****
good ↓ exit

AVG PRESS: 10.4
↑ exit

USAGE: 4.56hrs
↑ exit

These menus appear only if at least one Smart Data option has been enabled by the clinician.

SERVICING
enter ↑ exit

SN: 1234567890123
4567 ↓ exit

PCB: 123456789012
↑ exit

SW: SX432-0201
↑ exit